

Alco-Screen

Alco-Screen



ALCOHOL TESTING KITS

- Semi-quantitative results
- Convenient test procedure
- Results in two minutes
- Can test beverages for alcohol

ALCO
DRUG AND ALCOHOL TESTING PRODUCTS

1-800-227-9890

www.alcopro.com

Alco-Screen

Convenient, Fast Results

Testing with the Alco-Screen is as easy as placing one end of the test strip in the subject's mouth for a few seconds to wet it with saliva. The Alco-Screen requires such a small amount of saliva to activate the test even subjects with "dry mouth" can be tested successfully. Results are visible within two minutes after wetting with saliva.

Estimates Alcohol Concentration

Because the proportion of alcohol in saliva is directly related to the proportion of alcohol in blood, the Alco-Screen can easily and quickly detect the presence of alcohol and estimate intoxication levels. The presence of alcohol in saliva causes the reagent pad to turn shades of green. Higher concentrations of alcohol create darker shades of green. The operator estimates intoxication levels by comparing the color change against color standards printed on the foil package at the 0.02%, 0.04%, 0.08%, and 0.30% BAC.

Ideal for Zero Tolerance Testing

The Alco-Screen is ideal for agencies with zero tolerance policies. Any green color on the reagent pad after two minutes indicates the presence of alcohol of at least 0.02% or greater. 0.02% BAC is about the intoxication from one drink, and is typically the lowest cut-off level used to determine if a person has been drinking alcohol.

Detects Alcohol in Beverages

The Alco-Screen is one of the few devices that can detect the presence of alcohol in beverages. Because the Alco-Screen is calibrated to detect the very tiny amounts of alcohol present in saliva, the color standards on the test do not apply when testing beverages. The concentration of alcohol is extremely high in comparison to saliva. Dipping the strip into a beverage that contains alcohol causes the reagent pad to turn a very dark brown color.



SPECIFICATIONS

Test Medium: Saliva

Test Principle: Enzyme

Reading time: Two minutes

Results: Semi-quantitative

Measurement Range: 0.02%, 0.04%, 0.08%, 0.30% color standards

Shelf life: One year from date of manufacture



STEP 1

Place end of strip in mouth to collect saliva.



STEP 2

Compare the color change against the color standards on the back of the envelope package to estimate alcohol level.

Alco-Screen

Item	Description	Price
114	Box of 24 tests	\$40.00

For Information & Orders Call: **1-800-227-9890**
 Order 24 Hours A Day At: **www.alcopro.com**
 Fax Orders To: **1-800-655-5834**

ALCO PRO
 DRUG AND ALCOHOL TESTING PRODUCTS