

Q.E.D.

Q.E.D. Saliva Alcohol Test



ALCOHOL TESTING KITS

- DOT approved as Alcohol Screening Device
- Semi-quantitative results
- Accurate as a blood test
- Results in two minutes



1-800-227-9890

www.alcopro.com

Q.E.D.

Saliva Alcohol Test

DOT Approved as Alcohol Screening Device

The Q.E.D. was one of the first devices to be approved by the U.S. Department of Transportation (DOT) as an Alcohol Screening Device. DOT approval means that the Q.E.D. reliably and accurately detects alcohol at the 0.02 concentration or higher, and does not give false positive readings when no alcohol is present.

Semi-quantitative Results

Alcohol in the saliva creates a column of vivid reddish-purple color. The user reads the device at the upper limit of the color using the scale, exactly like reading a thermometer. If no alcohol is present no color develops.

Accurate as Blood Alcohol Test

Because the proportion of alcohol in saliva is directly related to the proportion of alcohol in blood, the Q.E.D. device can accurately measure intoxication levels. Studies have demonstrated a .98 correlation (1.00 is a perfect correlation) between saliva test results with the Q.E.D. and laboratory blood alcohol results.

STT Training Package

DOT regulations require operators to be certified as a Breath Alcohol Technician (BAT) or a Screening Test Technician (STT) to perform DOT regulated alcohol testing with the Q.E.D. device. The DOT approved STT Training Kit provides a cost-effective and convenient way to accomplish this required certification training.

SPECIFICATIONS

Test Medium: Saliva

Test Principle: Enzyme

Reading Time: Two minutes

Results: Semi Quantitative

Measurement Range: 0.01% - 0.15% BAC

Shelf Life: 12 months from date of manufacture



STEP 1

The subject holds the swab in the mouth for approximately 60 seconds to collect oral fluid.



STEP 2

The operator presses the swab into the test well until oral fluid fills the column.

RELATED SUPPLIES

Item	Description	Price
338	Q.E.D. Ethanol control, 1 bottle	\$20.00

Q.E.D. Saliva Alcohol Test

Item	Description	Price
110	Q.E.D.	\$7.25 @ 1-99 \$7.00 @ 100+
410	Q.E.D. STT Training Package Includes: Facilitor Guide, one Student Workbook, 10 Q.E.D. devices, positive control, one STT certificate	\$145.00

For Information & Orders Call: **1-800-227-9890**
 Order 24 Hours A Day At: **www.alcopro.com**
 Fax Orders To: **1-800-655-5834**

ALCOPRO
 DRUG AND ALCOHOL TESTING PRODUCTS