

ETG ALCOHOL METABOLITE TEST

EtG Urine Alcohol Test



ALCOHOL TESTING KITS

- Onsite test works like an instant drug test
- Results within 5 minutes
- 500 ng/ml cut-off level minimizes false positives
- Ideal for abstinence monitoring
- Longest window of detection for alcohol use

ALCOPRO
DRUG & ALCOHOL TESTING PRODUCTS

1-800-227-9890
www.alcopro.com

EtG Urine Alcohol Test



ETG Alcohol Metabolite Test

The EtG alcohol metabolite test offers the longest window of detection for alcohol of any test method available...up to 80 hours after consumption. The EtG alcohol metabolite test is for forensic use only and is great for abstinence enforcement programs.

Fast and Easy to Use

This instant urine test works just like any instant drug test kit. The collector or tester dips the test strip into the specimen cup for 10-15 seconds and lays the test strip flat. After several minutes, lines will develop. A control line first develops to indicate the test kit is working correctly. If a line forms at the Test indicator, it is considered to be a negative test. The presence of the EtG metabolite at or above the cut-off level will prevent a line from forming at the Test indicator; the absence of a line at the Test indicator is read as a positive test.

What is EtG?

Ethyl glucuronide (EtG) is a direct metabolite of ethyl alcohol and is a marker for recent alcohol exposure. EtG may be detectable in the specimen as soon as 2 hours after drinking and may be detected for up to 80 hours after consumption.

What about false positives?

The cut-off level for the EtG test is 500 ng/ml. This eliminates most false positives that may occur from incidental environmental exposure to alcohol based products.

Specifications

Test Format: Urine

Medium: Dip Test

Cut Off Level: 500 ng/ml

ETG Alcohol Test

Item	Description	1-3 Boxes	4 - 9 Boxes	10+ Boxes
8DETG	Alcohol Urine Dip Test, box of 25	\$118.75 ea	\$115.20 ea	\$111.65

EASE OF USE

Step 1

Remove cover and dip the tip into specimen for 10 - 15 seconds.



Step 2

Replace cap



Step 3

Read results



For Information & Orders Call: **1-800-227-9890**
 Order 24 Hours A Day At: www.alcopro.com
 Fax Orders To: **1-800-655-5834**

ALCOPRO
 DRUG & ALCOHOL TESTING PRODUCTS