


Orawell Saliva Drug Test

SALIVA DRUG TEST

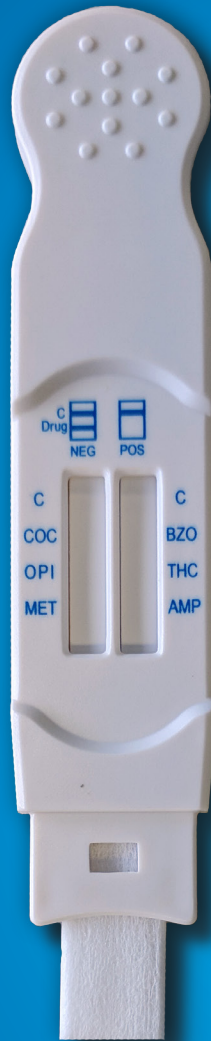

DRUG TESTING KITS

- Available in 6 and 12 panels
- Requires minimal saliva sample
- Easy, one-step collection procedure
- Meth panel detects ecstasy
- An observed collection process greatly reduces adulteration or substitution issues
- Orawell with alcohol testing panel is also available

ALCOPRO
DRUG & ALCOHOL TESTING PRODUCTS

1-800-227-9890
www.alcopro.com

DRUG TESTING KITS

Orawell Saliva Drug Test



Orawell Oral Fluid Drug Test

The Orawell 6 oral fluid drug test provides preliminary qualitative test results. Only a small amount of saliva is required to activate the test, reducing the testing time. To collect a saliva sample, the subject rubs the inside of the cheeks to stimulate saliva flow and then holds the device under the tongue. The collection process takes only a few minutes.

Observed Collections...Reduced Cheating

Because the subject collects the saliva in the presence of the operator, every collection is an Observed collection. This virtually eliminates the problems of adulteration and substitution common to urine drug testing. Saliva testing also eliminates the same-gender requirement for observed urine drug testing.

Interpreting Test Results

A red colored line should be observed in the control region marked C to indicate the test is performing properly. If no red line appears in the control region, the test is invalid and should be discarded and the operator should perform another test using a new oral fluid test. A red colored line observed in the test region indicates a presumptive negative test result. The color and density of the red line may vary in the control and test regions. A presumptive positive test result occurs if no red line appears in the test region.

The Orawell oral fluid test provides qualitative preliminary results, which only indicates the presumptive presence of the drug in someone's system. It does not measure intoxication levels.

Specifications	
Test Medium	Oral Fluid
Format	
Cutoff Levels	
Coc (Cocaine)	20 ng/mL
Meth (Methamphetamine) Includes Ecstasy	25 ng/mL
THC (9-tetrahydrocannabinol)	20 ng/mL
Amp (Amphetamines)	25 ng/mL
OPI (Opiates)	10 ng/mL
BZO (Benzodiazepines)	10 ng/mL
Fent (Fentanyl)	10 ng/mL
BAR (Barbiturates)	25 ng/mL
Oxy (Oxycontin/Oxycodone)	40 ng/mL
BUP (Buprenorphine)	10 ng/mL
MTD (Methadone)	30 ng/mL
TRA (Tramadol)	25 ng/mL

6-Panel Orawell Fluid Drug Test Kit

Item	Description	1-49	50-149	150+
821	THC/coc/opi/amp/meth/bzo	\$8.00 ea	\$7.80 ea	\$7.60
821-ALC	THC/coc/opi/amp/meth/bzo/ALC	\$9.00 ea	\$8.80	\$8.60

12-Panel Orawell Fluid Drug Test Kit

Item	Description	1-2 Boxes	3-5 Boxes	6+ Boxes
822	THC/coc/opi/amp/meth/bzo/bar/oxy/bup/mtd/fent/tra Box of 25	\$273.75	\$269.00	265.50

For Information & Orders Call: **1-800-227-9890**
 Order 24 Hours A Day At: **www.alcopro.com**
 Fax Orders To: **1-800-655-5834**

ALCOPRO
 DRUG & ALCOHOL TESTING PRODUCTS