


# ALCOHOL SCREENING DEVICE (ASD)

## Alco-Screen 02

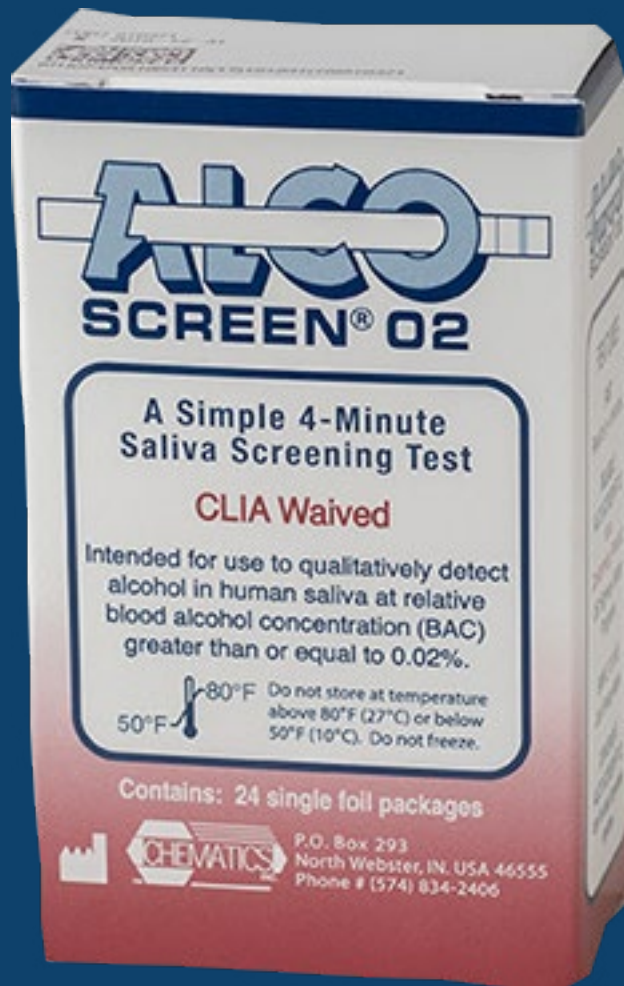

### ALCOHOL TESTING KITS

- Pass/Fail results at 0.02 alcohol concentration
- Convenient, Fast Results
- DOT approved as Alcohol Screening Device
- 24 test kits per box
- CLIA Waived for professional use

**ALCOPRO**  
DRUG & ALCOHOL TESTING PRODUCTS

1-800-227-9890  
[www.alcopro.com](http://www.alcopro.com)

## ALCOHOL TESTING KITS

# Alco-Screen .02



### DOT Approved as Alcohol Screening Device

The Alco-Screen 02 was one of the first devices to be approved by the US Department of Transportation (DOT) as an Alcohol Screening Device. DOT approval means that the Alco-Screen reliably and accurately detects alcohol at the 0.02 concentration or higher, and does not give false positive readings when no alcohol is present. DOT approval allows users to use the Alco-Screen 02 for screening tests required by Dept of Transportation.

### Convenient, Fast Results

To use the Alco-Screen 02, have the subject place the test strip into their mouth for 5 seconds until the reagent pad is saturated with their saliva. Then lay the test strip on a flat surface and start a four minute timer. After four minutes, read the results.

### Alco-Screen 02 Gives Pass/Fail Results

Because the proportion of alcohol in saliva is directly related to the proportion of alcohol in blood, the Alco-Screen can easily and quickly detect the presence of alcohol. Alcohol concentrations of 0.02 or greater cause a solid green line to form on the reagent pad. If no alcohol is present the reagent pad remains unchanged.



**Ask about online Screening Test Technician or STT Instructor training and certification.**

## EASE OF USE

### Step 1

Have the subject insert the test strip into their mouth for :5 seconds to wet the test pad.



### Step 2

Lay the strip on a flat surface and start a 4 minute timer.

### Step 3

After 4 minutes, read the results. A green or blue colored line on the pad means the device detects the presence of alcohol at the .02% level or higher. If no line appears, then there is no alcohol present.

### Specifications

Test Medium:	Saliva
Measurement Range:	0.02% - pass/fail

### Alco-Screen .02

Item	Description	Price
113	Box of 24	\$ 45.00 box
ctrl	Controls	\$ 35.00

For Information & Orders Call: 1-800-227-9890  
Order 24 Hours A Day At: [www.alcopro.com](http://www.alcopro.com)  
Fax Orders To: 1-800-655-5834

**ALCOPRO**  
DRUG & ALCOHOL TESTING PRODUCTS