

SALIVA TEST

Alco-Screen



ALCOHOL TESTING KITS

- Semi-quantitative results
- Convenient test procedure
- Results in two minutes
- Test beverages for alcohol

ALCOPRO
DRUG & ALCOHOL TESTING PRODUCTS

Alco-Screen



Convenient, Fast Results

Testing with the Alco-Screen saliva alcohol test is fast and easy. The Alco-Screen requires a small amount of saliva to activate the test, even subjects with dry mouth can be tested successfully. Results are visible within two minutes.

Estimates Alcohol Concentration

Because the proportion of alcohol in saliva is directly related to the proportion of alcohol in blood, the Alco-Screen can quickly detect the presence of alcohol and estimate intoxication levels. The presence of alcohol in saliva causes the reagent pad to turn shades of green. Higher concentrations of alcohol create darker shades of green. The operator estimates intoxication levels by comparing the color change against color standards printed on the foil package at the 0.02%, 0.04%, 0.08%, and 0.30% BAC.

Ideal for Zero Tolerance Testing

The Alco-Screen is ideal for agencies with zero tolerance policies and for non-regulated employee workplace testing programs. Any green color on the reagent pad after two minutes indicates the presence of alcohol in the saliva of at least 0.02% or greater. A 0.02% BAC is approximately the intoxication level from one drink, and is typically the lowest cut-off level used to determine if a person has been drinking alcohol.

Alco-Screen Detects Alcohol in Beverages

The Alco-Screen can detect the presence of alcohol in beverages. Because the Alco-Screen is calibrated to detect the very tiny amounts of alcohol present in saliva, the color standards on the test do not apply when testing beverages. The concentration of alcohol is extremely high in comparison to saliva. Dipping the strip into a beverage that contains alcohol causes the reagent pad to turn a very dark brown color.

EASE OF USE

Step 1

Instruct the subject to wet the test pad with their saliva for several seconds. Once the reagent pad is saturated with the saliva, place the strip on a flat surface. Set a 2 minute timer.



Step 2

After 2 minutes compare the color change against the color standards on the back of the envelope package to estimate alcohol level.



Specifications

Test Medium:	Saliva
Measurement Range:	0.02%, 0.04%, 0.08%, and 0.30% color standards

Alco-Screen

Item	Description	Price
114	Box of 24	\$40.00 box

For Information & Orders Call: **1-800-227-9890**
 Order 24 Hours A Day At: **www.alcopro.com**
 Fax Orders To: **1-800-655-5834**

ALCOPRO
 DRUG & ALCOHOL TESTING PRODUCTS