

CheckPoint

CheckPoint Breath Alcohol Test



ALCOHOL TESTING KIT

- Easy to use crush and blow alcohol test
- Detects alcohol at low concentrations
- Quickly provides Pass/Fail test results
- Multiple alcohol concentration cut-off levels available

ALCOPRO
DRUG & ALCOHOL TESTING PRODUCTS

1-800-227-9890
www.alcopro.com

CheckPoint



Easy-To-Use, Crush and Blow Test Kit

The CheckPoint breath alcohol test is an easy-to-use, disposable crush and blow breath alcohol test that provides results in only 2-minutes. The CheckPoint is an ideal alcohol screening test device where zero tolerance alcohol programs need to be enforced.

Fast, Pass/Fail Results

The CheckPoint can detect alcohol concentrations as low as 0.02% BrAC, which is about the intoxication level from one drink for most people. The user squeezes the device's tube and then blows directly into the tube for 12 seconds. If alcohol is present in the breath at the designated cut-off level, the yellow crystals turn to a green or blue color, which indicates a positive test result. If no alcohol is present in a subject's breath, the color of the crystals remain yellow and the subject is considered to have passed the test.

Multiple Cut-Off Levels

The CheckPoint breath alcohol test comes in a variety of cut-off levels to meet the specific needs of workplace, corrections, military, substance abuse or other areas where safety and compliance is important and where individuals must be monitored for alcohol use. Available cut-off levels include 0.02%, 0.04%, and 0.08% BrAC.

SPECIFICATIONS

Test Medium:	Breath
Test Principle:	Dichromate chemical
Reading Time:	Two minutes
Results:	Qualitative Pass/Fail
Cut-Off Levels:	0.020%, 0.040%, 0.080%
Shelf Life	Three years from manufacturer's date

EASE OF USE



Step 1

Crush the test ampule between your fingers.



Step 2

Have the subject blow into the end of the test ampule for 12 seconds.



Step 3

At 2-minutes, observe the color in the test ampule. A blue or green color indicates the presence of alcohol at the designated cut-off level.

ITEM	DESCRIPTION	1-24	100+
CCP02200	0.020% cut-off threshold	\$2.35	\$2.05
CCP04200	0.040% cut-off threshold	\$2.35	\$2.05
CCP08200	0.080% cut-off threshold	\$2.35	\$2.05

For Information & Orders Call: **1-800-227-9890**
 Order 24 Hours A Day At: **www.alcopro.com**
 Fax Orders To: **1-800-655-5834**

ALCOPRO
 DRUG & ALCOHOL TESTING PRODUCTS