

iCup

iCUP



DRUG TESTING KITS

- Multiple drug panel configurations available
- Test card built into collection cup
- Secure screw-on lid
- Flat front enables photocopying results
- Results in five minutes
- Temperature strip on cup

ALCOPRO
DRUG & ALCOHOL TESTING PRODUCTS

1-800-227-9890
www.alcopro.com

DRUG TESTING KITS

iCup



No-mess Collection Procedure

The iCup uses an integrated cup design that builds a drug detection card securely into a urine specimen container to minimize the collector's contact with the urine specimen. The donor provides a urine specimen directly into the iCup and attaches the leak proof screw-on lid. The integrated cup design eliminates the need to pipette the urine onto a test card or to dip a test strip into the urine. All the iCup collector does is peel back the label to view the test results.

Photocopy Test Results

The unique flat front design of the iCup enables the operator to photocopy test results to create a permanent record. We include a photocopy report template so that collectors can document donor identification and other relevant details of the collection process and the test results.

Easy Laboratory Confirmation

Collectors can send the leak proof iCup container containing the urine specimen directly to a laboratory to easily confirm non-negative results. Sending the original specimen in the iCup avoids the need to pour the specimen into another container, saving time and avoiding additional handling of the urine specimen. Using the security seals included with the iCup creates a strong chain of custody that documents from the point of collection to sending the specimen to the laboratory.

SPECIFICATIONS

Test Medium: Urine

Format: Integrated specimen cup with temperature strip

Cut-off levels:

AMP (amphetamine)	1,000 ng/ml
BAR (barbiturates)	300 ng/ml
BZO (benzodiazepines)	300 ng/ml
COC (cocaine)	300 ng/ml
MDMA (Ecstasy)	500 ng/ml
THC (marijuana)	50 ng/ml
MTD (methadone)	300 ng/ml
mAMP (methamphetamine)	1,000 ng/ml
MOR (morphine)	300 ng/ml
OPI (opiates)	2,000 ng/ml
OXY (oxycodone)	100 ng/ml
PCP (phencyclidine)	25 ng/ml
PPX (propoxyphene)	300 ng/ml
TCA (tricyclic antidepressants)	1,000 ng/ml

EASE OF USE



Step 1

Temperature strip assures valid specimen.



Step 2

Peel back label to view vivid, dark result lines.



Step 3

Leak-proof lid allows operator to lay specimen on it's side for photocopying results.

Item	Description	1 - 24	Box of 25 Tests
8i33*	THC/coc/mAMP	\$5.60	\$120.00
8i41*	THC/coc/mAMP/opi	\$6.95	\$148.75
8i53*	THC/coc/mAMP/opi/amp	\$7.95	\$173.75
8i81	THC/coc/mAMP/opi/amp pcp/bzo/bar	\$10.95	\$248.75
8i101	THC/coc/mAMP/opi/amp pcp/bzo/bar/mtd/tca	\$12.50	\$281.25

* Denotes products are CLIA Waived

Adulteration panel available on 4 panel, 5 panel, 8 panel and 10 panel iCup test kits.

For Information & Orders Call: **1-800-227-9890**
Order 24 Hours A Day At: **www.alcopro.com**
Fax Orders To: **1-800-655-5834**

ALCOPRO
DRUG & ALCOHOL TESTING PRODUCTS