

# Nicotine Test

## Urine Nicotine Test



### URINE NICOTINE TEST KIT

- Economical nicotine test
- Accurately identifies tobacco users
- Quickly provides Pass/Fail test results
- Ideal for smoking cessation and insurance programs

**ALCOPRO**  
DRUG & ALCOHOL TESTING PRODUCTS

1-800-227-9890  
[www.alcopro.com](http://www.alcopro.com)

# Rapid Nicotine Test



## Accurately Identifies Tobacco Users

The rapid Nicotine test uses a cut-off level of 200 ng/mL of cotinine to accurately identify tobacco users. Cotinine is the major metabolite of nicotine and is the preferred marker of nicotine use because its half-life is much longer than nicotine. The presence of cotinine is a reliable indicator of tobacco use. Because the rapid nicotine test uses a relatively high cut-off level, you can be confident that individuals with positive results are tobacco users.

## Fast, Pass/Fail Results

The rapid nicotine test is a qualitative test that gives pass/fail results. The presence of cotinine at or above the cut-off level prevents a line from forming in the test region. The absence of a line in the test region indicates a positive result. If cotinine is not present at the cut-off level, a line forms in the test region, indicating the test is negative.

## Ideal for Smoking Cessation Programs

The rapid urine nicotine test offers the most economical method to detect tobacco use. The accuracy of the test makes it ideal for smoking cessation and insurance programs. It is also useful in tobacco-free environments to verify an individual's self-report of tobacco use.

## EASE OF USE



### Step 1

First allow the test kit to come to room temperature before use.

### Step 2

Using the dropper, add 3 drops of urine into the specimen well marked "S"

### Step 3

Read the test results at 5 minutes. The presence of a line in the control region marked C indicates the test is performing properly. The presence of a line in the test region indicates a negative test result.

## SPECIFICATIONS

Test Medium:	Urine
Format:	Cassette and dropper
Cut-Off Levels:	200 ng/mL

ITEM	DESCRIPTION	1-39	40+
900	Rapid urine nicotine test	\$2.00	\$1.75

For Information & Orders Call: **1-800-227-9890**  
Order 24 Hours A Day At: **www.alcopro.com**  
Fax Orders To: **1-800-655-5834**

**ALCOPRO**  
DRUG & ALCOHOL TESTING PRODUCTS